
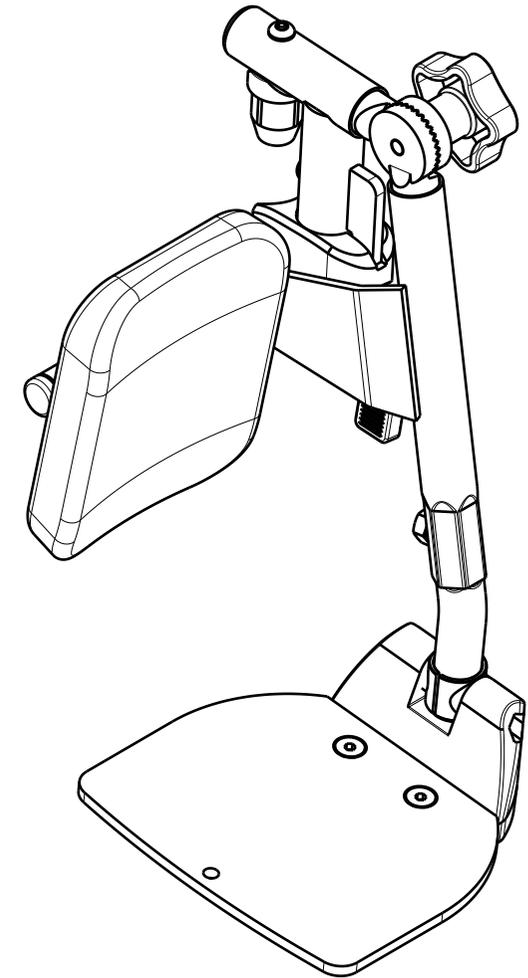


Only left side depicted.

LEGREST BZ8K / ECLIPS X4 KIDS			
Scale:	1:3	Drafted	Verified
Date	26.08.2022	02/10/2025	02/10/2025
Initials	KS	AZ	AS

Plan No.
E00042447/C1.1

No.	Plan	Article	Quantity
1	P00041794(L)	2657454	1
	P00042443(R)	2657554	1
2	P00042667	2657600	1
3	Y006924(L)	1603054	1
	Y006188(R)	1603154	1
4	Y002210	1389754	1
5	P00107670 (28)	2909654	1
	Y005844(32)	1547154	1
	P00030402 (37)	2649654	1
6	XC00183	3992471	1
7	XP00007	2990000	1
8	X001443	1389254	1
9	X003779	1900228	1
10*	X002282	1992854	1
11*	X002170	1992792	1
12*	X002164	1488200	1
13	X002138	1992416	1
14*	X002126	1992429	1
15*	X002124	1992404	1
16*	M5 DIN985	2994006	1
17	M6x45 ISO7380	1991017	1
18	Cap D22	1990648	1
19	M8x10 DIN916	1008942	1
20	M6x16 ISO7380	1008459	2
21	M6x20 ISO7380	1006999	1
22	D6,4 DIN7349	1003321	2
23	M6x45 DIN912	1003109	1
24	M6 DIN985	1001680	5
25	Cap D16	1001598	1
26	M8	1001581	1
27*	D5,3 DIN125A	1001550	1
28	M6x16 DIN7991	1994752	2

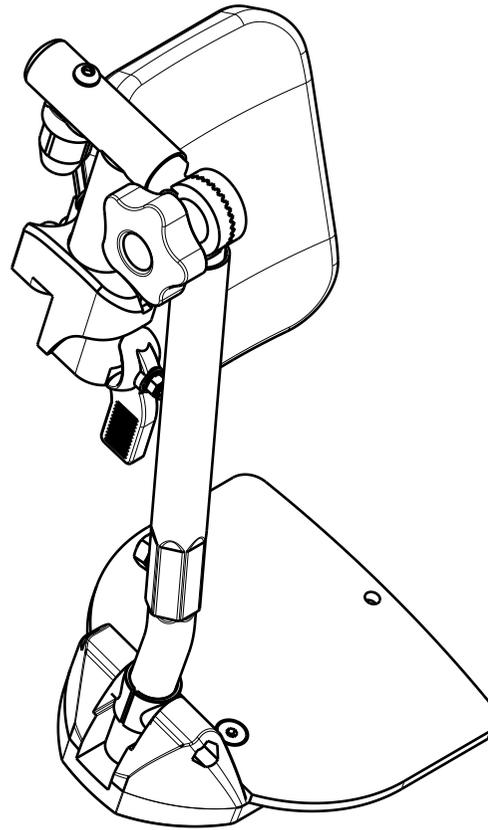

1995423* - LOCKING SYSTEM COMPLETE			
No.	Plan	Article	Quantity
10*	X002282	1992854	1
11*	X002170	1992792	1
12*	X002164	1488200	1
14*	X002126	1992429	1
15*	X002124	1992404	1
16*	M5 DIN985	2994006	1
27*	D5,3 DIN125A	1001550	1

* - subparts of 1995423

LEGREST BZ8K / ECLIPS X4 KIDS			
Scale:	1:3	Drafted	Verified
Date	26.08.2022	02/10/2025	02/10/2025
Initials	KS	AZ	AS

Plan No. E00042447/C1.1
